

**stryker**

**SV2**

# Hospital Bed

**Customisable to fit your needs**



### **Versatility is its greatest feature**

At Stryker, creating a hospital bed like our SV2 takes collaboration. By working together with medical professionals like you all over the world, we are delivering a high quality bed you can customise to your specific needs.



**Mobile headboard**

Allows more ergonomic positioning for caregivers and enhanced patient benefits in any bed position.

**Integrated control panels**

The intuitive controls offer great functionality to both caregiver and patient in any bed position.

**Double regression**

Allows 89 mm backrest and 39 mm knee gatch regression to help reduce both sacral pressure and displacement of the patient.



**"Always on" under bed light**

Enhances patient visibility during the night when exiting the bed, helping to prevent falls.

**5th wheel**

Fifth wheel mobility enhances the manoeuvrability of the bed, helping to reduce caregiver back injury. The fifth wheel allows the bed to go in a straight direction and facilitates mobility in small spaces.

**Fold down siderails**

Simplifies patient exiting, enhances patient safety by fully covering and minimising entrapment gaps, and ease of use for caregivers who can tuck away the siderails with only one hand.



250 kg safe working load



10 year life expectancy



IEC 60601-1-2:2014



Low bed indicator

**The SV2 hospital bed successfully combines form and function to enhance the well-being and comfort of patients while helping caregivers work efficiently. Always keeping patient and caregiver safety at the forefront:**



### **Low bed indicator**

The low bed indicator clearly indicates when the bed is in its lowest position, helping to reduce patient injury from falls. During the night, caregivers have a clear overview of the beds so they can ensure that all are in their lowest position.

### **Smart egress**

Our focus is to prevent patient falls. The smart egress function raises the backrest and brings the bed surface to its easiest exiting height. Patients can exit the bed with support from the safety siderail.



### **Safety siderails**

Patient safety is our number one priority. The SV2 fold down siderails facilitate patient exiting and enhance safety by fully covering the mattress base. The siderails can be lowered individually with just one hand. The design minimises entrapment gaps and allows easy and profound cleaning.

### **Integrated siderail controls**

The SV2 is equipped with dual siderail controls so both patient and caregiver can easily operate the bed. The nurse facing siderail unit allows caregivers to lock patient controls and use the electronic CPR function in case of an emergency.



### **Patient pendant**

Every patient deserves to be in the most comfortable position. The patient pendant allows patients to easily adjust the bed position to their needs.

### **Nurse control unit**

Healthcare professionals require flexibility to move around the bed and give their patients the required care. The nurse control unit allows control of the bed from different locations. Equipped with electronic CPR functionality, Trendelenburg, anti Trendelenburg and examination position, care for your patients can be provided from different locations.



**The one-touch controls allow patients and caregivers to easily adjust bed positioning, thus eliminating the need for manual labour. Offers an ergonomic design to both patients and caregivers.**

**Versatile.** Throughout the episode of care.



**The SV2 hospital bed is built to be a versatile bed for any hospital by considering the needs for both efficiency and safety. It is fully adjustable for both the caregiver's ease of use and the patient's comfort.**

The great range of height adjustment, 37 to 75 cm, brings the bed close to the ground to prevent patients from falling and supports an ergonomic working position for the caregivers.

Taller patients benefit from a bed extender, extending the bed with an additional 31 cm. The associated linen tray is a built-in storage

unit located at the footboard, that can store patients' clothes, laundry and the nurse control unit at the footboard.

The 5th wheel provides enhanced steering and turning, potentially reducing caregiver back injuries and particularly useful in confined spaces.



The mobile headboard allows a more ergonomic working position for caregivers. The head- and footboard are easily removable to allow a caregiver quick access to their patient.

CPR levers are located on both the left and right side of the backrest. The levers are easily accessible with raised as well as lowered siderails.

Activating the CPR release positions the bed at 0°, necessary for effective CPR.

The always-on under bed light enhance patient visibility when getting out of bed at night, helping to prevent patient falls.



250 kg safe working load



Bed Extender  
31 cm



Height adjustment  
37 – 75 cm

**Customisable.** To fit your needs.

**Healthcare should be customised to accommodate patient, family and caregiver needs. That's why Stryker offers a diverse array of products and services that help improve health and well-being of both patients and caregivers.**

### **SV2 Custom**

The SV2 is a feature rich and value driven bed that includes standard features like a low bed indicator, dual siderail controls and convenient one-touch electronic controls. The SV2 custom is customisable with optional features.

**1, 2, 3, 4, 5, 6 & 7 (option to select multiple features)**



### **SV2 Package Plus**

Offering a bed that fits your needs is important to us. In addition to the SV2 Custom, we offer two extra packages to support the ease of decision making. The Package Plus includes all standard features plus dual wheel castors, a fifth wheel and the nurse control unit.

**Included options: 1, 2 & 5**



### **SV2 Package Complete**

The SV2 Complete is our fully equipped model including all of the available options to meet your needs.


**Included options: 1, 2, 3, 4, 5, 6 & 7**



### **Options in a row**

- |                                       |  |
|---------------------------------------|--|
| <b>1</b> 5th wheel                    | <b>5</b> Dual wheel castors                              |
| <b>2</b> Nurse control unit           | <b>6</b> Under bed light                                 |
| <b>3</b> Patient pendant              | <b>7</b> X-ray translucent backrest with cassette holder |
| <b>4</b> Bed extender with linen tray |  |



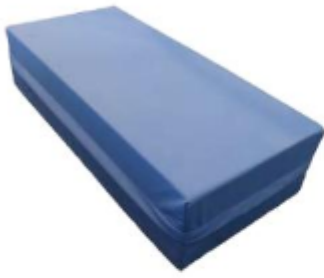

**Bolster for bed extension**

**12 cm bolster**

(D) 33cm x (W) 71cm x (H)18 cm

Net weight: 1,4 Kg

**14 cm bolster**

(D) 33cm x (W) 71cm x (H) 20 cm

Net weight: 1,6 Kg



**Stainless steel IV pole**


Height adjustable

**Weight capacity:**

2Kg per hook

**Width:** 42 cm

**Height:** 107 cm – 176 cm

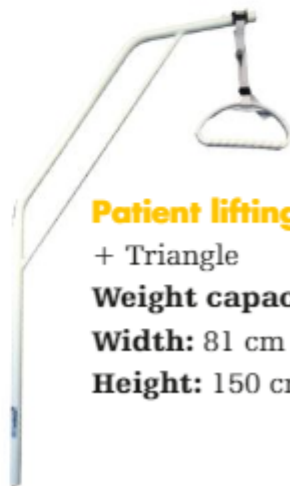

**Patient lifting pole**

+ Triangle

**Weight capacity:** 75Kg

**Width:** 81 cm

**Height:** 150 cm



**Foley bag basket**

203 mm x 198 mm x

139 mm [WxHxD]



**Oxygen bottle holder**

275 mm x 390 mm x

145 mm [WxHxD]

**Compliant to international bed standards**

- IEC 60601-1 Ed. 3.1: 2012
- IEC 60101-1- 6 Ed. 3.1: 2013
- IEC 62366 Ed. 1.1:2014
- IEC 60601-2-52:2009 + A1:2015
- IEC 60601-2-54 Ed. 1.1: 2015
- IEC 60601-1-2:2014 Ed. 4.0

**Bed colours**



Stryker Blue:  
Pantone 534C



White:  
(RAL 9002)



Silk Grey:  
(RAL 7044)



Pastel Blue:  
(RAL 5024)



Yellow Signal:  
(RAL 1003)

**3 wooden options**



Wooden



Maple



Walnut

The **SV mattress series** are designed to comfort patients during hospital stay. The **SV mattresses** are intended to assist in the prevention and treatment of pressure ulcers.

**Foam series mattresses**

Mattress options	Size	Fire barrier options
12 cm standard foam	200 x 86 x 12	<ul style="list-style-type: none"> <li>•None</li> <li>•EN597-1, EN597-2, BS6807, UNI9175</li> </ul>
14 cm standard foam	200 x 86 x 14	<ul style="list-style-type: none"> <li>•None</li> <li>•EN597-1, EN597-2, BS6807, UNI9175</li> </ul>

**Properties**

- Durable and cost-effective patient support
- Adaptable to most bed frames
- Turnable
- High resilience and monodensity foam
- Conforms to all bed positions



**Foam mattress**


**Viscoelastic foam series mattresses**

The viscoelastic foam surface mould to the patient's shape, evenly distributes the weight and contours the body in any position, increasing comfort and pressure redistribution.

Mattress options	Size	Fire barrier options
12 cm viscoelastic foam	200 x 86 x 12	<ul style="list-style-type: none"> <li>•None</li> <li>•EN597-1, EN597-2, BS6807, UNI9175</li> <li>•BS7177</li> </ul>

**Properties**

- Temperature sensitive
- Reduces interface pressure
- Waterproof cover
- Conforms to body shape
- Multi-stretch cover
- Vapour permeable

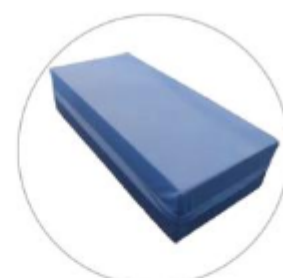

**Bolster mattress**



**Visco mattress**

**High quality products and services are part of the Stryker commitment to customer satisfaction. Our ProCare services solution provides the highest standard of service and support for your facility's needs.**

**ProCare services**

With ProCare services, our highly trained technicians help ensure your equipment is ready to perform when needed. ProCare is a partnership you can count on, to give you one less thing to worry about and one more reason to feel confident you're doing all you can for your staff and patients.



# Flex Financial

Through Stryker's Flex Financial business, we provide a range of smart alternatives designed to fit your organisation's needs. We collaborate with our customers for assistance in finding the financial solution that suits your needs\*.

**Making a difference.** Reducing our impact.

We consider the well-being of our planet to be as significant as the health of the individuals who live on it.



**Both hazardous and non-hazardous waste is recycled by our facility.**



**Use of environmental friendly painting chemicals.**



**ISO 14001:2004 Environmental Management System compliance**

\*Program availability will vary depending on country and product portfolio. Please check with your local sales representative for full details and availability.

## Medical

**This document is intended solely for the use of healthcare professionals.**

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product.

Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker, SV2. All other trademarks are trademarks of their respective owners or holders.

The products depicted are CE marked in accordance with applicable EU Regulations and Directives.

This material is not intended for distribution outside the EU and EFTA.



16251EN

OTHMED-BRO-60\_16251EN

DLS 02/2018

2017-16251